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保养检测项目 PM Check List (Service Contract Customers Only)	周期 Period	执行结果 Result	备注及后续维护 Comments/Follow up
系统 System			
检查系统文件 Review Error Logs	Quarterly		
电气环境监测 Electrical Environment Check (Input Voltage/Room Temperature/Room Humidity)	Quarterly		
测试扫描架指示灯和紧急制动按钮 Test Display Lights and E-Stop Buttons	Quarterly		
测试扫描架防挤压传感器 Test Patient Tilt Sensor	Quarterly		
测试 E-off 开关 Test Emergency off SW if SEO installed at site	Quarterly		
检查合同中规定的由 GE 负责维修的 UPS UPS Check per SVC Contract, UPS Power off/Power on Inspection	Quarterly		
备份系统数据 Confirm Save State Completed	Quarterly		
检查 x 射线曝光指示灯 Check X-Ray on indicators	Quarterly		
检查各扫描按钮操作正常 Scan control pushbuttons test	Quarterly		
更新场地维修记录 Update site log	Quarterly		
检查系统时间的准确性 Check System Clock	Quarterly		
检查 Insite 的联通状态 Check insite connectivity	Quarterly		
扫描测试 System Scanning Test,	Quarterly		
图像质量检测 IQ check with phantom	Yearly		
校准 (如需要) Calibration (if required)	Yearly or as required		
检查 GE 提供的屋顶固定装置(如有) Ceiling Component Bolts Inspection (if supplied by GE)	Yearly		

心脏扫描功能检查 Cardiac Option Inspection	Yearly		
操作台 OC			
清理监视器 Console Monitor Cleaning	Quarterly		
清理滤网、风扇和通气栅格 Clean Components Filters, Fans & Grills	Quarterly		
操作台检查和清理 Clean, Inspect, Replace Desktop Components	Quarterly		
检查操作台风扇 Check all fans is operational	Quarterly		
检查防震固定装置(如需) Seismic Anchor Check (if necessary)	Quarterly		
清洁 SCIM/键盘/鼠标/轨迹球 Clean SCIM / keyboard/ trackball / mouse	Quarterly		
外设操作功能检查 Operational functional check for MOD/DVD drivers / SCIM / keyboard/ trackball / mouse	Quarterly		
检查和清理 HP 工作站的内部风扇 Check/clean the fans inside host computer (Video Card/CPU/GPU/DIMM)	Yearly		
扫描架 Gantry			
检查并记录扫描架旋转圈数 Check the number of Gantry revolutions RTSViewer -D	Quarterly	____万圈	
扫描架轴承加油润滑 (圈数每隔二百万) Grease main bearing (上次加油后, 滑环需要再次加油的累计次数为: _____万圈)	Per 2M Revolution		
检查扫描架运转时有无异常噪音 Listen for unusual noise	Quarterly		
检查前后盖和顶盖线缆的固定 Gantry Cover Cables Inspection	Quarterly		
清除滑环刷子粉末 Remove Slip Ring Brush Debris	Quarterly		
环面轨道检查 Slip Ring Track Inspected	Quarterly		
清理刷块 Clean Brush Blocks	Quarterly		
检查刷头磨损 Brush Tip Inspection	Quarterly		
清理数据采集系统滤网 Cleaning DAS Detector Plenum Filter	Quarterly		
清理数据采集系统风扇 Cleaning DAS Detector Plenum Fans	Quarterly		
清除探测器表面灰尘 Cleaning Detector Face Plate	Quarterly		
清理扫描架加热器滤网 Clean Gantry Heater Filter	Quarterly		
清理顶盖风扇 Clean Top Cover Fans	Quarterly		
检查旋转部分的部件和电缆没有松动和刮擦 Inspect Rotating Path Components and Cables	Quarterly		
检查并清洁扫描窗 Scan Window Inspection/Clean/Replacement	Quarterly		
检查 DAS 风扇运作情况 Inspect Plenum 5 Fans Operational	Quarterly		
检查旋转部分与扫描架之间的间隙 Inspect Rotational Clearances	Quarterly		
用力矩扳手检查旋转部分的固定螺丝 Double check the fixed screws with torque wrench especially for the rot. Parts on Gantry	Quarterly		

定位灯位置检查 Laser Alignment Inspection	Yearly		
检查准直器功能 Aperture and filter positions and operation	Yearly		
检查准直器窗口 Inspect protective window and opening, clean debris	Yearly		
检查扫描架前/后倾速度 Verify that the tilt speed is 1degree/sec within +/-10degree	Yearly		
扫描架倾斜速度调整 Tilt Speed Adjustment	Yearly		
扫描架倾斜 30 度检查 Verify gantry can tilt to +/-30degree	Yearly		
检查扫描架 0 度角 Gantry Tilt Zero Check	Yearly		
检查/调整轴电机驱动皮带 Inspect and adjust tension belt	Yearly		
检查油泵/储油盘/液压杆/液压接头/油管是否漏油 Inspect the pump/reservoir/tilt cylinder/fittings for oil leak	Yearly		
检查编码器的橡胶齿条 Inspect Rotational Encoder Wheel	Yearly		
编码器齿条磨损过度, 需要更换 Encoder Replacement, Data Last Replaced: _____	Yearly		
高压部分 HV System			
检查并记录球管扫描次数 Check the number of Tube slices	Quarterly		
检查高压油槽绝缘油状态 HV oil check in tank receptacle	Quarterly		
清理/检查球管散热滤网 Inspect and clean tube heat exchanger	Quarterly		
清理/检查逆变器风扇 Inspect and clean JEDI inverter fan	Quarterly		
反馈电路检测 Meter Verification	Yearly		
高压油箱反馈电阻检测 HV Tank Feedback Resistor Verification	Yearly		
HHS 扫描 HHS Scan	Yearly		
电源分配柜 PDU			
通风口和 PDU 内部清理 Vacuum Covers and PDU interior	Yearly		
检查各器件有无变色和打火痕迹 inspect the PDU electronic components for dis-colorization and arcing damage	Yearly		
紧固各电源线的连接 Torque all Connectors	Yearly		
确认变压器抽头跳线与输入电压相符 Verify Tap Connection Match	Yearly		
检查防震固定装置(如需) Seismic Anchor Check (if necessary)	Yearly		
扫描床 Table			
检查和清洁扫描床 Inspect/Clean Table Pan	Yearly		
检查扫描床地角固定状况 Inspect Table Anchor	Yearly		
检查扫描床的液压器件有无漏油 Hydraulic fluid leak inspection	Yearly		
检查扫描床驱动皮带 Inspect driver belt	Yearly		
检查/清理扫描床驱动滚轮 Inspect/Clean cradle driver roller	Yearly		
润滑导轨 Grease Table Rail (cradle and IMS)	Yearly		

检查升降轴承的保持架 Inspect bearing retainer	Yearly		
检查升降轴承的润滑并按需补注润滑油 Verify/grease table leg bearing is greased	Yearly		
检查平板床的固定(如有) Flat Table Top Mounting Inspection	Yearly		
检查床盖 Inspect Table Covers	Yearly		
检查扫描床的附件 Inspect Cradle Accessory Attachment (Note in comments if anything is replaced)	Yearly		
测试防挤压开关 Check Tape Switches	Yearly		
检查扫描床的升降和与 Gantry 互锁 Table and gantry interference check (table min/max position at gantry -30/0/+30 degree)	Yearly		

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